



## Installation

The Flex PC app lets you control your Flex VRA system directly from your computer wirelessly and seamlessly alongside your audiometer software.

- Create custom hotkeys for VRA activation
- Use it simultaneously while audiometry software is being utilized.
- Completely wireless. Just plug and play!

### Step 1: Install

Locate the Flex VRA USB and run "Flex VRA Installer".exe. Flex PC will install on the computer and place a shortcut icon on the desktop called "Flex PC."



### Step 2: Plug In

Locate the Flex VRA USB Transmitter (PC Host) and insert it into the computer.



← "PC Host"

### Step 3: Launch

Double click the desktop Flex PC icon to launch Flex PC. **If the Flex PC window opens and the buttons respond, installation is complete.**

You can now control your Flex VRA:

- Activate Left, Right, and Center VRA
- Create custom hotkeys in Setup
- Adjust transparency to float over audiometer software

