



## Flex VRA Software Version 3.1 Update Instructions

Before beginning the update process, please make sure you have your new remote controls and update USB drive ready.

### Step 1

Remove any hardware or accessories connected to the bottom of the monitor so that all USB ports are available.

### Step 2

Locate one of the black USB receivers included with your new remote controls and insert it into an open USB port on the monitor.

### Step 3

Press and hold your finger on the top left corner of the screen for approximately three seconds, then release.

If the Service Menu does not appear:

1. Tap the screen once to open the on-screen menu
2. Select Setup

### Step 4

Inside the Setup menu, locate and select the Service button, then confirm the selection.

If you do not see a Service button:

- Press the Toy button on your new remote control to enter Service Mode

### Step 5

The monitor should now display a black Windows desktop screen.

Insert the update USB drive into one of the USB ports on the bottom of the monitor.

## Step 6

Windows File Explorer should open automatically.

If it does not:

- Select the folder icon on the taskbar at the bottom of the screen

If the taskbar is not visible:

- Swipe upward from the bottom of the screen to reveal it

## Step 7

In Windows File Explorer, locate and open the D: drive.

## Step 8

Inside the D: drive, locate the file named Install.

Press and hold your finger on the Install file for approximately three seconds, then release and select:

- Run as Administrator

## Step 9

A dialog box will appear asking whether this monitor is configured as a:

- Left Monitor
- Center Monitor
- Right Monitor

Select the appropriate option for your setup.

## Step 10

The update process will now begin and may take approximately 10 to 15 minutes to complete.

During this time:

- Do not press any buttons on the new remote controls
- Do not disconnect power
- Do not remove the USB drive

Once the update is complete, the system will automatically reboot back into Flex VRA.

Need Assistance?

For additional support, please contact Flex VRA.

<https://www.flexvra.com>

[sales@flexvra.com](mailto:sales@flexvra.com)

512-777-0016