

## HARDWARE



The Flex VROCA Button comes preprogrammed to communicate with the Flex VRA. Signal transmission is done wirelessly to any monitor where the USB dongle is inserted.

The Flex VROCA Button uses the kinetic energy of the button press to generate just enough electricity to transmit a wireless packet. This means there is no need for changing of batteries...ever!

## SETUP

On a button press, the monitor will activate for a specified amount of time. **The default setting is 7 seconds.** To alter the length between 2 and 10 seconds, access the service menu by pressing the top left corner of the monitor for 3 seconds. Tap "VROCA BUTTON" on the left-hand menu and select the desired timing from the dropdown. Press "UPDATE." Once complete, press "Play Flex VRA" at the bottom of the screen.

## OPERATION



### Single Button Press


A single press from the button will activate the monitor for the time specified in the service menu (2-10 seconds).



### Double Button Press

A quick double press of the button activates Dynamic Conditioned Play Audiometry. A static image (curtain, box, egg) appears. When the button is pressed, the image will animate to reveal a character.

Double tap the button or L, C, or R keys on the remote to return to VRA mode.

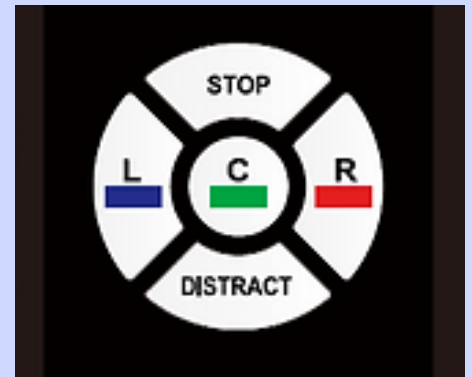

The "STOP" button on the remote suppresses activation from the button in the event the patient presses when not desired.

Double tap L, C, and R to toggle activation of Dynamic CPA and make a single press of L, C, or R mimic a button press activation.